



INSTRUCTIONS FOR URINE TEST

Broadlands Farm
101 Durkar Low Lane
Durkar
Wakefield
WF4 3BJ
01924 242851

Please read instructions completely before you begin.

1. Unscrew cap to container provided.
2. Do not remove preservative from container, the preservative is toxic if swallowed & irritating to eyes and skin. Antidote: rinse exposed area with water, then wash with soap and water. If swallowed, do not induce vomiting; give plenty of water to drink and seek medical attention.
3. To reduce contamination, the specimen submitted for analysis should be a clean catch "midstream" sample.
4. Provide 50 ML of urine in addition to the 40 ML of preservative into the container and re-screw the lid onto the container.
5. Fill out your name and date on label provided.
6. Mail back to PCI Europe on Thursday to arrive Friday, in the box provided.
7. Results will be ready within 7-10 business days of receipt of sample, not including weekends and holidays.

Please send test kit back in original packaging

If you have any questions please contact us on enquiries@parasitetesting.co.uk.